GB INSTRUCTION FOR USE

Read the instruction for use before using this medical device. Keep out of reach of children. Stop using the product immediately when skin irritation occurs. Test pad is recommended for those with high allergic tendencies. Use only on intact skin. For single use only. Store at room temperature. Avoid storing product in direct sunlight and/or high temperatures.

1. CONTENTS (REF 87-200037 / 87-200039)

1x Instruction for use
12x Sports tape, 2,5 cm x 10 m or
12x Sports tape, 3,8 cm x 10 m

CONTENTS (REF 87-200013 / 87-200028)

1x Instruction for use
1x Sports tape, 2,5 cm x 10 m or
1x Sports tape, 3,8 cm x 10 m

CONTAINS

Rayon; unbleached, Zinc oxide adhesive; rubber-based

2. DESCRIPTION

Athlete Sports Tape is widely used in sport and physiotherapy as a strapping tape. Preventative taping reduces the risk for injuries on ligaments. Athlete Sportst Tape is a powerfull adhesive tape that supports muscles, tendons and joints during sports activities.

FEATURES

- easy to tear
- strong adhesion
- allows skin perspiration

3. INTENDED USE / INDICATIONS FOR USE

Athlete Sports tape provides excellent comfort for muscles and joints. This sportstape is used to prevent and treat injuries for stabilisation and immobilisation.

- only on intact skin
- people over 12 years of age
- for single use only

4. INSTRUCTIONS FOR USE

Incorrect application of sports tape can cause injuries. Consult a qualified person for the correct application of the sports tape. Storage temperature can have influence on the adhesive strength of sportstape.

INSTRUCTION

- Thorough cleaning and drying of the skin is essential for the proper adhesion of all (sports) tapes.
1. Check if the skin is clean, dry and free from any creams or lotions.
 2. If body hair is limiting adhesion you may need to trim/shave (area should be hair-free).
 3. Cut the tape with scissor / or tear the tape by hand according to the necessary length.
 4. Apply it strip-by-strip and anchor the tape.
 5. Press it down to activate the adhesive.
 6. Apply the bandage with little or no stretch.

DO NOT

Apply it very tightly because of the risk of cutting off blood circulation.

REMOVAL

- avoid lifting the skin, never rip or pull off tape forcefully
- pull with the direction of hairgrowth and
- while removing the tape: immediately move the thumb (with a little pressure) to keep the skin down

CAUTION

Test pad is recommended for those with allergic tendencies. Stop using the product immediately when skin irritation occurs.

5. CONTRAINDICATIONS

DO NOT USE

- if sensitive to one of the ingredients or skin irritation occurs
- history of hyper-sensitive skin or pre-existing skin conditions e.g.; psoriasis, eczema, dermatitis
- on open wounds, active infection or irritation of area of skin to be taped
- in case of cancer, kidney disease, congestive heart failure (CHF), deep venous thrombosis (DVT)
- if circulation compromised in area

6. WARNINGS & PRECAUTIONS

Keep out of reach of children. Do not re-use. Do not apply directly on wounds.

7. COMPLAINTS & CUSTOMER SERVICE SUPPORT

For questions, complaints or reporting an adverse reaction, you can contact the manufacturer.

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